

<i>General description</i>	Waterproof nurse brooch fob watch. A nurse watch is not a medical device within the meaning of Directive 93/42/EEC	
<i>Attachments</i>	No	
<i>Batch reference</i>	No	
<i>Connected device</i>	No	Watch with silicone case
<i>Single use device</i>	No	
<i>Sterile device</i>	No	
<i>Sterilisation</i>	No	
<i>Autoclavable</i>	No	
<i>Cleaning</i>	Yes	Remove the watch from its silicone case and clean the silicone part with a cloth, using soapy water or 70% alcohol. For the watch itself, use a damp cloth (watch is waterproof).
<i>Component parts</i>	Yes	Watch with silicone case
<i>Material</i>	Silicone, metal, plastics, and glass watch crystal.	
<i>Contains latex</i>	No	
<i>Directive</i>	2004/108/EC and 2002/95/EC	
<i>Standard</i>	Complies with standards: EN 61000-6-1: 2001 and EN 61000-6-3: 2001 + A11: 2004	
<i>Labelling</i>	No	
<i>Packaging</i>	Yes	Metal case, bag, and small cardboard box. Protective foam inside.
<i>Special packaging</i>	No	
<i>Duration of use</i>	Continuous	
<i>Marking on device</i>	Yes	Comed logo
<i>References + extensions</i>	24 300 40-41-45-46-47-48-49-51-52-60-62-65 / 24 303 00	
<i>Designation</i>	Silicone nurse watch	
<i>Dimensions L x W (mm)</i>	85 x 40	
<i>Thickness (mm)</i>	10	
<i>Crystal ø (mm)</i>	25	
<i>Net weight (g)+1%</i>	28	
<i>Battery</i>	SR626SW button cell	
<i>Colours</i>	40 = White; 41= Blue; 45 = Black; 46 = Pink; 47 = Green; 48 = Purple; 49 = Red; 51 = Yellow; 42 = Orange; 60 = Zebra; 62 = Flowers; 65 = Leopard; 24 303 00 = white, blue, pink kit	
<i>Temperature resistance (°C)</i>	-20° to + 50°C	
<i>Usage</i>	For reading the time and noting beats per minute.	
<i>Instruction for use</i>	<p>To start the watch, detach it from its silicone case and remove the protective tab on the winder. Pull the winder and turn it to select the time, then push the winder back in to set. Replace the watch in the opposite side to the brooch fastener.</p> <p>Changing the battery: On the back of the watch there is a groove that allows the battery case to be opened. Lift the back part up with a thin, pointed object. Lift the white plastic cover and remove the battery by sliding it onto its side. Insert the new battery with the correct polarity (with the "+" upwards). Replace the white cover and close the battery case.</p>	

<i>Precautions for use and/or single use warning</i>	Be careful not to pierce yourself with the brooch pin. Do not leave exposed to the sun. You may notice that one of the hands moves loosely before the watch has been started. This is because the watch has not yet been activated. Once the protective tab has been removed, the hand will only be able to move in one direction.
<i>Recycling</i>	Yes
<i>Recovery</i>	Through an approved waste disposal company or your COMED distributor / Yes for the battery, via a used battery collector through your distributor. Also see the BATTERIES procedure at www.comed.fr
<i>WEEE/REACH</i>	Yes see WEEE procedure at www.comed.fr/ (SR626SW button) / Yes see BATTERIES
<i>Safety data</i>	Hazard symbols = none S- and R-phrases = none