## FR - [REF] 29 050 00 COFFRET LARYNGOSCOPE

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### OPERATION INSTRUCTION:

Batteries: 2 Alkaline Battery pcs required  
Engage the blade onto hook pin.  
Apply force upward to bring the blade in operating position.  
To bring the blade in stand by position apply force down word.

### CARE & MAINTENANCE OF FIBER OPTIC LARYNGOSCOPES BLADES

Great care should be taken when handling fiber optic blades and light guide. Don't drop the blades or unduly bend the Light guides otherwise the fibers inside will fracture reducing output of light.  
The blades should be gently scrubbed in soapy water with soft brush; this provides a through physical cleaning.  
After cleaning, through rinse the blades and dry at 65oC max.

 Warning: Ultrasonic cleaning is not recommended.  
STERILIZATION : The fiber optic are not autoclaveable. You can sterilize by E.O. Gas, with 65 oC max.

### CARE & MAINTENANCE OF LARYNGOSCOPES BATTERY HANDLE

Cleaning / Sterilization :  
Handle will with stand the same cold soak solution and autoclave ranges outlined in the blade section. However, the batteries and bulb must be removed out (there are not autoclaveable). The clean and sterilization temperature is 121 oC during 15 minutes minimum.

### BLADES and HANDLE TEST PROCEDURE

Laryngoscope Handle and Blades should always be tested after cleaning / disaffection / sterilization and prior to use.  
To test, connect the laryngoscope blade to handle and pull open to the ON position.  
If the unit fails to light or flickers, check the lamp / batteries and the electrical contacts.  
Be sure adequate supplies of spare lamps, batteries, and replacement parts are readily available.  
If problem still persists, contact supplier please.

 CAUTION: Federal law restricts this device to sale by or on the order of a physician.

For more informations, you can contact us by mail : [info@comed.fr](mailto:info@comed.fr)

Or by post:  
COMED  
PARC DES FORGES – 8 Rue Louise Michel  
67200 STRASBOURG – France

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STURDY INDUSTRIAL CO., LTD. 168, Sec 1,